

Post Procedure Enzymatic Z-Sponge

Order # ZUTR50560 – Round, Add Water

Order # ZUTR50562 – Flat, Add Water



Intended Use

For efficient manual 'point of use' cleaning to remove gross contaminants from endoscope and other soiled surgical instruments.

- Multi-enzymatic cleanser to break down fat, protein, blood and other bioburden
- Non-abrasive, neutral pH
- Safe for all medical devices and endoscopes
- Single use

Instructions for Use

1. Check sponge package to ensure it is within expiration date.
2. At point of use, tear open package and remove Z-sponge.
3. Place Z-sponge into a container of water up to 1000ml. Effective in water temperatures up to 64 C (150 F)
4. Squeeze Z-sponge 5-7 times in water to activate and mix enzymatic with water.
5. Squeeze Z-sponge and remove the sponge from the container, use Z-sponge to wipe down exterior of endoscope or surgical instrument.*
7. For flushable channels, place distal end into container and flush enzymatic solution through instrument per IFU.*

*Always clean per endoscope or surgical instrument IFU.



ZUTR50560



ZUTR50562