

Intended Use

- Semi-rigid expandable mesh protects the tip/lens of endoscopes from front and side impact. Can also be used with sensitive surgical instruments.
- For use during all transport and storage and is designed for one time usage
- Single use, do not reuse.
- Allows moisture to escape, eliminating potential bacteria growth.
- Non-abrasive/will not scratch any part of the instrument/lens.

Instructions for Use

1. Slide the **smaller** end of the tip protector onto the end of the instrument that requires protection immediately after the instrument has been high level disinfected.
2. Allow 20 – 30% of the **larger** end of the tip protector to overhang the end of the instrument.
3. Remove the tip protector just before procedure.
4. Place back on instrument after pre-cleaning process for transport to cleaning room.
5. Remove and dispose of tip protector prior to leak testing.

NOTE: Product is made of polyethylene material which is compatible with paracetic acid, hydrogen peroxide, glutaraldehyde & ortho-phthalaldehyde products.

Specifications & Ordering Information

- Small / Red Endoscope Tip Protector is for instruments that are approximately 2mm to 10mm
- Large / Blue Endoscope Tip Protector is for instruments that are approximately 7mm to 15mm
- Temperature range for the tip protector is -94° to +175° F; (-70° to +79° C). Tip protector is clean, not sterile.

Order # **ZUTR50512** – Small, Red - Qty / cs. 100

Order # **ZUTR50513** – Large, Blue - Qty / cs. 100