

Endoscope Cleaning Brushes

Order #'s ZUTR50510, ZUTR50511, ZUTR50522



Product Information

- Single-use brushes used to clean the channels and valves of flexible endoscopes.
- Cannot be reused or reprocessed.
- Consult the IFU from the endoscope manufacturer and the cleaning agent manufacturer.
- Dispose in accordance with facility policy.

Instructions for Use

1. Channel Cleaning Brush

- a) Select appropriate brush for the channel to be cleaned
- b) Always brush channels while they are submerged in the cleaning solution.
- c) Follow endoscope IFU for sequence of channel brushing.
- d) Use short strokes, always advancing the brush completely through the channel until it emerges from the appropriate end of the endoscope.
- e) Carefully advance the entire cleaning brush completely out of the end of the scope.
- f) Inspect whether there are debris on the bristles. If debris are observed, submerge the bristle end of the brush in the cleaning solution using gloved fingertips to remove the debris.
- g) Carefully pull entire brush out of channel.
- h) Re-inspect whether there are debris on the bristles. If debris are observed, submerge the bristle end of the brush in the cleaning solution using gloved fingertips to remove the debris.
- i) Visually inspect cleaning brush for kinks, torn bristles, or damage. Dispose of cleaning brush if any damage observed.
- j) Repeat step through every channel until no debris are observed on the bristles or per manufacturer IFU's.
- k) Use the brush and the cleaning solution of choice to clean reusable valves and buttons.
- l) Dispose of brush after use.

2. Valve Cleaning Brush

- a) Remove all valves and buttons from the endoscope.
- b) Use the valve cleaning brush and the cleaning solution of choice to gently cleanse all valve openings.
- c) Avoid twisting or torque of the brush handle. The Brush handle will prevent the brush from being pushed too deeply in the valve openings.
- d) Use the brush and the cleaning solution of choice to clean the control head and the control head dials.
- e) Use the brush and the cleaning solution of choice to clean reusable valves and buttons.
- f) Dispose of brush after use.