

Post Procedure Enzymatic Cleaning Kit

Order # ZUTR50507



Product Information

- Flexible Endoscope Bedside Cleaning Kit
- Ready to use Multi-Enzymatic Cleanser immediately breaks down and removes fat, protein, blood and other bioburden after a flexible endoscope procedure
- Contents include: 500ml of Multi-Enzymatic Cleanser and round Z-Sponge for cleaning exterior of scope
- Stable wide base for easy fluid suction

Instructions for Use

1. After completing procedure, open the package and wipe down insertion tube with the Z-Sponge.
2. Place the distal end of the endoscope into the pouch and suction the remaining solution through the biopsy/suction channel.
3. Cycle air and solution through channel.
4. Place scope into bag or container and send to decontamination area.

**Always clean in accordance with scope manufacturer's recommendations.*