

Concentrated Z-Sponge

Order # ZUTR50505



For efficient manual pre-cleaning to remove gross contaminants from endoscopes and other soiled surgical equipment.

Instructions for use:


1. Check the sponge packaging to ensure it is within the expiration date.
2. Tear open the package and add contents in 10 gallons of water or less. Squeeze sponge to ensure all enzymatic cleanser is added. Effective in water temperatures up to 64°C (150°F).
3. After use, dispose of sponge in accordance with your facilities rules.
4. Soak the instruments and scopes for a minimum of 2 minutes, heavy soil may require a longer soak time.
5. Rinse enzymatic cleanser from instruments and scopes prior to sterilization or high level disinfection.

Product Information:

- Non-Foaming Multi-Enzymatic Cleanser
- Safe for all medical devices and scopes
- Non-Abrasive, neutral pH
- Single use
- Non-foaming for optimal clarity

Order # ZUTR50505

Zutron Medical, LLC
17501 W 98th St #40-27
Lenexa, KS 66219
877-343-5873
www.zutron.com

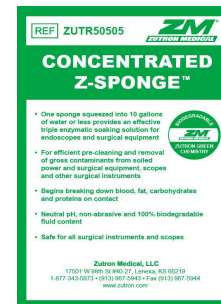

ZUT-Lab-50505IFU REV G