


Concentrated Multi-Enzymatic Cleaners

Instructions For Use: Manual Cleaning



For breaking down bioburden to facilitate the decontamination of soiled medical instruments, devices and endoscopes.

1.

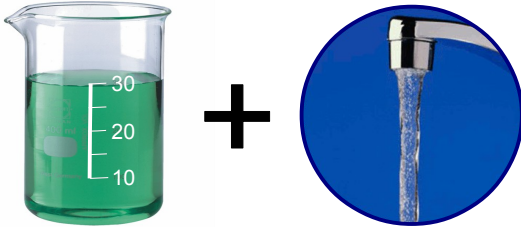

Dilute ½ – 1 ounce of Enzymatic Cleanser per 1 gallon of water.

2.




Soak instruments for a minimum of 2 minutes or longer as necessary.

3.



After soaking rinse bioburden and cleanser from instruments.

4.

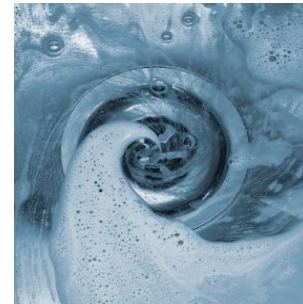

After each use dispose of and mix a fresh solution.

Cleanser Information and Precautions

- Effectively breaks down blood, proteins, fat, carbohydrates and other particles.
- Pleasant fragrance fights odors on contact.
- Safe for all surgical instruments, endoscopes and other health care equipment.
- Non-abrasive, neutral pH and 100 % biodegradable formula.
- Not to be used as a disinfectant.
- Cleanser must be rinsed from instruments prior to sterilization.
- Do not freeze or use at temperatures above 64°C.
- Expires 2 years after date of manufacture. Expiration date is stamped on the side, near the bottom of the bottle.