



Daily Calibration Log for Endo-Leak Tester (ZUTR30005)

Endo-Leak Tester is compatible with ACMI, Fujifilm, Fuse, Olympus, Pentax, Richard Wolf & Storz endoscopes.

Calibrate the ENDO-LEAK TESTER daily:

1. Connect the Calibration Gauge to the Endo-Leak Tester by utilizing the Calibration Gauge Adapter.
2. Turn the Endo-Leak Tester on Press "RUN"
3. The Endo-Leak Tester is calibrated to produce air pressure at 150 mmHg.
4. If the gauge shows +/- 3mmHg than your desired pressure, please contact Zutron Medical

Month/Year: _____

Serial #: _____

DAY	PASS	FAIL	INITIAL	COMMENTS
1	PASS	FAIL		
2	PASS	FAIL		
3	PASS	FAIL		
4	PASS	FAIL		
5	PASS	FAIL		
6	PASS	FAIL		
7	PASS	FAIL		
8	PASS	FAIL		
9	PASS	FAIL		
10	PASS	FAIL		
11	PASS	FAIL		
12	PASS	FAIL		
13	PASS	FAIL		
14	PASS	FAIL		
15	PASS	FAIL		
16	PASS	FAIL		
17	PASS	FAIL		
18	PASS	FAIL		
19	PASS	FAIL		
20	PASS	FAIL		
21	PASS	FAIL		
22	PASS	FAIL		
23	PASS	FAIL		
24	PASS	FAIL		
25	PASS	FAIL		
26	PASS	FAIL		
27	PASS	FAIL		
28	PASS	FAIL		
29	PASS	FAIL		
30	PASS	FAIL		
31	PASS	FAIL		

Please refer to the Endo-Leak Tester IFU for more details. Or contact your local distributor or Zutron Medical for further assistance: 877-343-5873