

Endo-Flush Unit Competency



Endo-Flush Quality Assurance Log

Always follow endoscope manufacturer's IFU pertaining to reprocessing of endoscope and accessories.

The Endo-Flush is to be used only during manual cleaning process of endoscope. The Endo-Flush is never to be used to disinfect an endoscope.

User: _____

Date: _____

Reviewer: _____

Clinical Skill	Met	Date	Comment	Initials
1. Locates Endo-Flush instructions for use manual on Zutron website				
2. Correctly adjusts flushing time.				
3. Correctly performs daily flow volume verification test and documents results.				
4. Demonstrates correct tubing connections to Endo-Flush and endoscope.				
5. Verifies need for fresh cleaning solution with each endoscope being reprocessed and proper dilution of cleaning solution.				
6. Correctly performs decontamination procedure and documents in a log.				
7. States recommended decontamination solutions per IFU.				
8. Verbalizes proper storage of tubing.				
9. Verbalizes how to report and label faulty equipment in their facility.				
10. Locates Safety Data Sheets for chemicals used.				
11. User wore PPE equipment: Gown, gloves and face shield.				

To complete unit competency, user must meet each line item. Reviewer initial each line once completed

User signature once completed: _____

Reviewer signature once completed: _____

Please refer to the Endo-Flush manual for more details. Or contact your local distributor or Zutron Medical for further assistance: 877-343-5873