

Flow Volume Verification Log



Endo-Flush Quality Assurance Log

1. Place the open end of the Flow Verification Tube into an empty 500ml or more graduated cylinder.
2. Adjust the timer to 15 seconds.
3. Press the "RUN/STOP/ENTER" button to begin the test and the timer display will begin counting down and fluid will begin flowing into the graduated cylinder.
4. Once the timer counts down to zero, flow will stop.
5. Remove the Flow Verification Tube from the graduated cylinder and measure how much fluid was dispensed.
6. If the volume is greater than 300 mls, then Endo-Flush is ready to use.
7. If the volume is not greater than 300mls, please contact your local distributor or Zutron Medical for further assistance.

Month/Year: _____

Endo-Flush Serial #: _____

Date	Flow Validation Test		Amount	Initials
	Pass	Fail		
1	Pass	Fail		
2	Pass	Fail		
3	Pass	Fail		
4	Pass	Fail		
5	Pass	Fail		
6	Pass	Fail		
7	Pass	Fail		
8	Pass	Fail		
9	Pass	Fail		
10	Pass	Fail		
11	Pass	Fail		
12	Pass	Fail		
13	Pass	Fail		
14	Pass	Fail		
15	Pass	Fail		
16	Pass	Fail		

Date	Flow Validation Test		Amount	Initials
	Pass	Fail		
17	Pass	Fail		
18	Pass	Fail		
19	Pass	Fail		
20	Pass	Fail		
21	Pass	Fail		
22	Pass	Fail		
23	Pass	Fail		
24	Pass	Fail		
25	Pass	Fail		
26	Pass	Fail		
27	Pass	Fail		
28	Pass	Fail		
29	Pass	Fail		
30	Pass	Fail		
31	Pass	Fail		

Please refer to the Endo-Flush manual for more details. Or contact your local distributor or Zutron Medical for further assistance: 877-343-5873