

# RTU Foaming Z-Sponge

Order # ZUTR50504



For efficient manual pre-cleaning to remove gross contaminants from endoscopes and other soiled surgical equipment.

## Instructions for use:

1. Check the sponge packaging to ensure it is within the expiration date.
2. Tear open package and remove sponge. Ready for immediate use – do not dilute in water or mix with other cleansers.
3. Wipe surface to be cleaned until all bio burden has been removed
3. After use dispose of sponge in accordance with your facilities rules.
4. Rinse enzymatic cleanser from instruments / scopes prior to sterilization or high level disinfection.

## Product Information:

- Contains RTU Multi-Enzymatic Foaming Cleanser
- Safe for all medical devices and scopes
- Non-Abrasive, neutral pH
- Single use

Order # ZUTR50504

Zutron Medical, LLC  
9816 Pflumm Road  
Lenexa, KS 66215  
877-343-5873  
www.zutron.com

