

**andorate**<sup>®</sup>

**AIR/WATER, SUCTION,  
BIOPSY VALVES &  
aquapulse**<sup>®</sup>  
by andorate

**AUXILIARY WATER  
CONNECTOR**

*Instructions for Use.*

*Read carefully prior to use.*



GA Health Company Limited

Unit 18, 21/F, Metropole Square,

2 On Yiu Street, Shatin, N.T., Hong Kong

Tel +852 2833 9010

www.GAhealth.net

info@GAhealth.net

**Not Made with natural rubber latex.**

**Does not contain DEHP**

<u>Part Number</u>	<u>Description</u>
GAR087A	Suction Valve
GAR087B	Air/Water Valve
GAR087C	2-piece valve kit (Suction, Air/Water Valves)
GAR087	3-piece valve kit (Suction, Air/Water and Biopsy Valves)
GAR064	Auxiliary Water Connector
GAR092	4-piece valve kit (Suction, Air/Water, Biopsy Valves and Auxiliary Water Connector)

For Fujifilm® 700 Series Endoscope

**INTENDED USE**

The single use **ANDORATE® Air/Water Valve** is used to control the air / water function of an endoscope during GI endoscopic procedures. The air / water valve in the endoscopic system provides backflow prevention function to the air / water channel. Not using the air / water channel can cause potential contamination to the air / water system.

The single use **ANDORATE® Suction Valve** is used to control the suction function of an endoscope during GI endoscopic procedures.

The single use **ANDORATE® Biopsy Valve** is used to cover the opening to the biopsy/suction channel of GI endoscopes. The biopsy valve provides access for endoscopic device passage and exchange, helps maintain insufflation, minimizes leakage of biomaterial from the biopsy port thought the endoscopic procedure and provides access for irrigation.

The single use **AQUAPULSE® Auxiliary Water Connector** is used in conjunction with the irrigation tubing, intended to provide irrigation via irrigation fluid such as sterile water supplied to Fujifilm® GI endoscope during gastrointestinal endoscopic procedures when used in conjunction with an irrigation pump. The Auxiliary Water Connector is manufactured with a one-way valve to minimize the risk of cross-contamination of the irrigation system.

**WARNINGS AND PRECAUTIONS**

- Follow the Fujifilm® guidelines for air/water/suction use in the Fujifilm® endoscopes
- GA Health Company Limited did not design these devices to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of these devices to ensure patient and/or user safety.
- These disposable medical devices are not intended for reuse. Any institution, practitioner, or third party, who reprocesses, refurbishes, remanufactures, reesterilizes, and or reuses these disposable medical devices must bear full responsibility for their safety and effectiveness.
- Endoscopic procedures should only be performed by persons having adequate training and familiarity with endoscopic techniques.
- Consult the medical literature relative to techniques, contraindications, complications and hazards prior to any endoscopic procedure.
- When handling these devices use appropriate aseptic technique.

**ANDORATE® Air/Water & Suction valves:**

- The air / water valve and suction valves are intended for single patient use only. These devices should be discarded after being used on one patient. It should not be used on multiple patients.
- Make sure the holes in the air/water and suction valves are not clogged or blocked. (This could cause continuous air/CO<sub>2</sub> flow, or inadequate suction.)

**ANDORATE® biopsy valve:**

- Do not use a sharp or pointed object to prime the valve prior to use.
- Do not leave a device hanging from the valve. Doing so can cause the creation of a larger valve slit/hole that may cause leakage.
- If the lid of the valve is opened while attached to the endoscope during a procedure, scope suction will be compromised and leakage may occur.
- If leakage occurs, a sterile gauze should be used to cover the valve.

- The device should be discarded after being used on one patient. It should not be used on multiple patients

**ANDORATE® Auxiliary Water Connector:**

**(Applicable to GAR064, GAR092)**

- Never attach or reattach the Auxiliary Water Connector to an endoscope that is/was inside the patient and has not been reprocessed.
- Do not insert this device into the patient. The device should be attached to the auxiliary water port of a flexible gastrointestinal endoscope and an irrigation source. Use proper techniques to avoid risk of aspiration of a patient.
- Always prime the auxiliary water channel or biopsy channel prior to inserting the endoscope into the patient.
- The device is part of distal irrigation system with a backflow prevention feature and is intended for single patient use only. The device should be discarded after being used on one patient. It should not be used on multiple patients.

**DIRECTION FOR USE**

1. Prior to clinical use, familiarize yourself with the device and read all the Instructions for Use.
2. Inspect the package for shipping or handling damage. If damage is evident, **DO NOT USE THESE DEVICES, SAVE THEM OR RETURN, AND CONTACT YOUR LOCAL PRODUCT SPECIALIST.**

**ANDORATE® Suction Valve:**

3. Remove the suction valve from the tray.
4. Insert the suction valve into suction cylinder of an Fujifilm® endoscope (see Figure 1).
5. Twist valve slightly to seat in proper orientation within the cylinder.
6. Confirm that the valve fits securely and that there is no gap between the valve end cap and the endoscope.  
**Note:** The valve should not be able to be rotated once it is fully inserted into the cylinder.
7. With the thumb fully on the suction valve button, depress fully downward and release as required.  
**Note:** For optimal performance, please soak the suction valve in the sterile water for one minute prior to use.

**ANDORATE® Air/Water Valve**

8. Remove the air/water valve from the tray.
9. Insert the air/water valve into the air/water cylinder of an Fujifilm® endoscope (see Figure 1).
10. Confirm that the valve fits securely and that there is no gap between the valve end cap and the endoscope.
11. Depress and release the valve to check if the valve moves smoothly and freely returns to its OFF position.  
**Note:** For optimal performance, please soak the air/water valve in the sterile water for one minute prior to use.

**ANDORATE® Biopsy Valve**

12. If using the 3-piece or 4-piece valve kit (See Figure 3), attach the Biopsy Valve.
13. Securely place the biopsy valve onto the biopsy/suction channel opening of a Fujifilm® endoscope (See Figure 1).  
**Note:** This biopsy valve does not accommodate a luer-lock syringe without an irrigating adaptor.
14. Close the lid when there is not a need to pass a device through the channel.
15. When the device diameter over 3.2 mm, please open the lid and pass the device through the hole into the biopsy/suction channel.

**ANDORATE® Auxiliary Water Connector  
(Applicable to GAR064, GAR092)**

16. If using the 4-piece valve kit (See Figure 4), attach the auxiliary water connector to the auxiliary water port by screwing it onto the port. When attaching the auxiliary water connector to the port, take care to ensure that the auxiliary water connector is aligned to the port and avoid excessive tightening. Misalignment or excessive tightening of the auxiliary water connector could lead to the possibility of leakage. Once the auxiliary water connector is attached to the auxiliary water port of the endoscope, attach Irrigation Tubing Set to the auxiliary water connector. For use of the Irrigation Tubing, reference Instructions for Use of it. (See Figure 2)
17. In the case there is any leakage due to misalignment or excessive tightening, please replace with a new auxiliary water connector.
18. Discard the auxiliary water connector after each patient use.

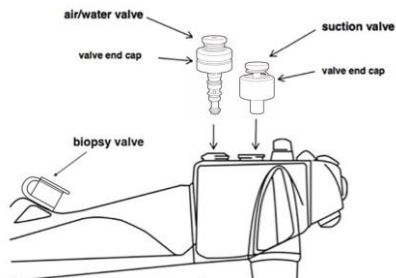


Figure 1

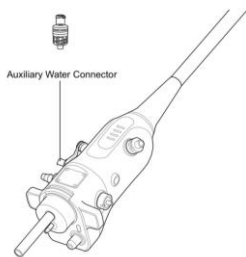


Figure 2

**3-piece-valve kit**

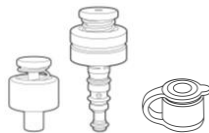


Figure 3

**4-piece valve kit**

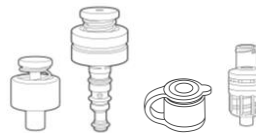


Figure 4

**PRODUCT DISPOSAL**

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

**WARNING**

An issued or revision date for these instructions is included for the user's information. In the event that two years have elapsed between this date and product use, the user should contact GA Health Company Limited to determine if additional information is available. Report any serious incident that has occurred in relation to the device to the distributor or manufacturer or competent authority of the member state.

ANDORATE® is a registered trademark of GA Health Company Limited.  
 AQUAPULSE® is a registered trademark of GA Health Company Limited.

Fujifilm® is a registered trademark of Fujifilm Corporation.

**EXPLANATION OF SYMBOLS USED ON LABELS AND INSTRUCTIONS FOR USE**

- Use-by date
- Batch or lot code
- Manufacturer
- Date of Manufacture
- Medical Device
- Catalogue number  
Product Code
- Sterilized using Ethylene Oxide
- Notified body TÜV SÜD
- Do not re-use  
Single use
- Do not re-sterilize



Product not made with natural rubber latex



Does not contain DEHP



Consult Instructions for Use



Authorized representative in the European Community



Do not use if package is damaged



Keep dry



Keep away from sunlight



Federal law (U.S.A.) restricts this device to sale, distribution and use by or on the order of a physician.