

# Liquid Instrument Detergent

## Instructions for Use



**For effective cleaning of medical devices and surgical instruments**

### Manual cleaning instructions

1. Pour Liquid Instrument Detergent into basin or sink at a dilution of 1/8 to 2 ounces of product per 1 gallon of water.

2. Dip instruments into solution for 2-5 minutes once instrument soils have been loosened. Dried on soils may require extra soak time.

3. Rinse thoroughly with water to remove instrument residues.

4. Wrap instruments and proceed with further processing. Solutions should be discarded daily.

### Automated washer instructions

1. Dilute Liquid Instrument Detergent in automated washer at 1/8 to 2 ounces of product per 1 gallon of water.

2. Water temperature should be based on equipment manufacturer's guidelines and recommendations.

3. Make sure machine goes through both wash and rinse cycles.

4. Always consult equipment manufacturer guidelines for recommended dilutions and other pertinent information.

### Cleanser Information and Precautions

- Safe for all surgical instruments, endoscopes and other general healthcare equipment
- Wear gloves and eye protection when using this product
- Non-corrosive, contains no chlorines, phosphates, or hydroxides
- Not to be used as a disinfectant
- Do not use at temperatures above 120°F
- Do not mix with other chemicals
- Do not allow cleanser to freeze