

Zutron Medical would like to make your day easier with the Endoscope Stiffening Device



Product Specifications

Dimensions:

1.4mm x 1700mm (Order # ZUTR141700)

1.6mm x 1700mm (Order # ZUTR161700)

1.6mm x 2000mm (Order # ZUTR162000)

% Increase of Scope Stiffness:

1.4mm Wire - 22% Average Increase

1.6mm Wire - 29% Average Increase

Design

Aluminum Anodized Handle

Nitinol Memory Wire

Warranty:

One year



Product Features

- **Makes difficult colons easy to maneuver**
Reach the cecum faster and reduces recurrent looping.
- **Shortens Procedure Time**
Prevents repositioning of the patient which eliminates laborious stress for nurses.
- **Reusable Device**
Prevents Reprocessable and long lasting design.
- **Compatible with most flexible scopes**
Compatible with Pentax®, Fujinon® and non-variable stiffness Olympus® endoscopes.
- **Easy and safe to use**
Engineered tapered design is safe and will not damage scope channels.

Product Description

When added scope stiffness is desired to negotiate a difficult colon, and prior to repositioning the patient, simply insert the Colonoscope Stiffening Device into the biopsy port to stiffen the insertion tube and advance to the cecum. The device is then withdrawn and the procedure is completed as usual.

These reusable devices are compatible with Pentax®, Fujinon®, and Olympus® non-variable stiffness endoscopes.

Ordering Information

Order #	Description
ZUTR141700	Colonoscope Stiffening Device (Standard)
ZUTR161700	Colonoscope Stiffening Device (Firm)
ZUTR162000	Enteroscope Stiffening Device (Firm)