



ValveSafe Basket Directions for Use

Order # ZUTR50519

Description

Single-use valve cage for the safe storage of endoscope valves and to ensure that they remain as part of a unique set with the parent endoscope, as recommended by a number of national and international guidelines.

Instructions

1. Remove the valves from the endoscope.
2. Manually clean as recommended by the valve manufacturer.
3. Place the valves into the open Valvesafe. The chain of the A/W cleaning valve should pass through a recess.
4. Close the top of the Valvesafe around the endoscope's umbilical cord, ensuring the four pins are fully located into the opposite housings.
5. Ensure that the Valvesafe is closed around the umbilical cord and not in the scope insertion tube.
6. Reprocess the endoscope, and place in drying cabinet if necessary.
7. Pull the "ring pull" to open the Valvesafe and to release the valves.
8. Place the valves back with the endoscope.
9. Safely discard the Valvesafe. **Do not re-use.**

Specifications

- Single-use – cannot be reused.
- "Smart-Click" technology ensures cage is 'broken' and therefore prevents re-use.
- Locks onto the parent endoscope – without concealing any part.
- Takes multiple valves, including cleaning valves.
- Plastic construction to prevent 'rust' and prevent any microbiological growth.
- Large openings to allow solutions through.

The polymer used for the construction of Valvesafe is approved for endoscopy use. There should be no reaction with:

- Enzymatic detergents
- Alkaline detergents

Compatible with disinfectants typically used for endoscope reprocessing, including peracetic acid, glutaraldehyde, OPA, etc.