



## Valve Bag - Directions for Use

Order # ZUTR50520

### Description

This product is designed to help ensure the traceability of these critical endoscope components after reprocessing through use by keeping the valves and accessories with the scope.

### Instructions

1. Reprocess flexible endoscope and accessories following the IFU of the device manufacturer.
2. Open mesh bag.
3. Place the small endoscope valves and accessories inside the mesh bag.
4. Tie the bag securely onto the endoscope (around the biopsy port if possible).
5. Single-use only, do not reuse.

### Specifications

- Order # **ZUTR50520**
- Case of 100
- Tough, lightweight
- Green mesh bag.