

*REVISED INSTRUCTIONS FOR USE

RTU Enzymatic Wipes Order # ZUTR50506



For efficient manual pre-cleaning to remove gross contaminants from soiled power and surgical equipment, scopes and other surgical instruments.

Product Information

- Pre-soaked with Multi-Enzymatic Foaming Cleanser RTU
- Safe for all surgical instruments and scopes, non-abrasive, neutral pH
- 2 year shelf life
- Approved for use – 1 wipe per 500mls of water makes effective detergent solution
- Single use, 100 wipes per canister / 6 canisters per case

Instructions for Use – Suctioning Detergent and Wiping Down Endoscope

After completing procedure, use no less than 1 wipe diluted in 500mls of water to make an effective enzymatic detergent solution to suction through an endoscope.

1. Soak and squeeze no less than 1 wipe in 500mls of water to create the enzymatic detergent solution.
2. Use the wipe to wipe down the exterior of the endoscope.
3. After wiping the exterior of the endoscope, dispose of wipe in accordance with your facility's procedures.
4. Place the distal end of the endoscope into the 500ml enzymatic detergent solution and suction the fluid through the endoscope.
5. Follow facility procedures for transportation of the endoscope to decontamination area.

Instructions for Use – Wiping Down Endoscope Only

1. Remove cover and seal from container and pull wipe through opening. Replace cover and pull wipe out at an angle.
2. Wipe is ready for immediate use – do not mix with other cleansers.
3. Unfold wipe and use to wipe down the exterior of the endoscope until all bio-burden has been removed.
4. Repeat if necessary.
5. After wiping the exterior of the endoscope, dispose of wipe in accordance with your facility's rules.
6. Follow facility procedures for transportation of the endoscope to decontamination area.

Zutron Medical, LLC
9816 Pflumm Road
Lenexa, Kansas 66215
Phone 877-343-5873 Fax 913-967-5943